

EXERGEN

TemporalScanner

Instructions for Use

Disposable Caps for TAT-5000S and TAT-2000 series of the Exergen TemporalScanner™ Thermometer

Disposable Caps are designed for the Exergen TemporalScanner™ Thermometer and can be used to prevent contamination between patients.

During the temperature measurement session:

Single-use: The Disposable Caps can be used once on a single patient then discarded in regular trash.

DO NOT reuse the same cap for different patients. Using the same cap for multiple patients can increase the risk of infection.

Multiple-use: The multiple use of Disposable Caps is restricted to a single patient only.

Note: To ensure optimal hygiene, it is recommended to use antiseptic disposable wipes to clean the Exergen TemporalScanner™ Thermometer.

Using the Disposable Caps:

Figure 1



Figure 2



Figure 3

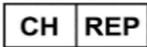


Apply cap by pushing onto the probe head with fingers. After temperature measurement is completed, remove the cap by pushing edge forward with thumb. (Figure 2)

EXERGEN

TemporalScanner

If a serious incident has occurred in relation to this device, it should be reported to the manufacturer (service@exergen.com) and the competent authority of the Member State in which the user and /or patient is established.

	Exergen Corporation 400 Pleasant Street Watertown, MA 02472 USA		Do not re-use
	Consult instructions for use		Single patient multiple use
	Medical device		Conformite Europeenne
	Keep dry		Keep away from sunlight
	MedEnvoy Switzerland Gotthardstrasse 28 6302 Zug Switzerland		Emergo Europe Westervoortsedijk 60 6827 AT Arnhem The Netherlands
	Emergo Consulting (UK) Limited Compass House Vision Park Histon c/o Cr360 – UL International Cambridge CB24 9BZ England, United Kingdom	  	INMETRO



www.exergen.com/caps