EXERGEN

TEMPOTHERM™

TEMPORALSCANNER™ TAT-2000 SERIES

INSTRUCTIONS FOR USE
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
GEBRAUCHSANWEISUNG
MODE D'EMPLOI
ISTRUZIONI D'USO
GEBRUIKSAANWIJZING
BRUKSANVISNING
KUUMEMITTARIN KÄYTTÖOHJEET
BRUKSANVISNING
BRUGSANVISNING
KULLANIM TALÏMATLARI
UPUTE ZA UPOTREBU
Οδηγίες χρήσης
הוראות שימוש
تعليمات الاستخدام

WWW.INTERNATIONAL.EXERGEN.COM

EXERGEN

TEMPOTHERM™

TEMPORALSCANNER™ TAT-2000 SERIES

+31 6 53226285

TEMPOTHERM@EXERGEN.COM

WWW.INTERNATIONAL.EXERGEN.COM

EXERGEN CORPORATION
400 PLEASANT STREET
WATERTOWN, MA 02472, USA

ENGLISH	05
ESPAÑOL	09
PORTUGUÊS	13
DEUTSCH	17
FRANÇAIS	21
ITALIANO	25
NEDERLANDS	29
SVENSKA	33
SUOMI	37
NORSK	41
DANSK	45
TÜRKÇE	49
HRVATSKI	53
Ελληνικά	57
עברית	61
اللغة العربية	65

HOW TO USE THE EXERGEN'S TEMPOTHERM™ THERMOMETER

This information is about using Exergen's Tempotherm™ thermometer for measuring body temperature and/or a possible fever. These instructions should help you use the thermometer safely and effectively.


Scan this QR-code to read the Technical User Manual of your Tempotherm™ Temporal Artery Thermometer.

Or go to international.exergen.com/tempotherm for the Technical User Manual of your Tempotherm™ thermometer.

FAMILIARISE YOURSELF WITH THE INSTRUMENT

- Push and hold the button to scan for temperature.
- Beeping and LED lights show temperature changes.
- Reading stays on display for 30 seconds.
- To restart, hold the button again.

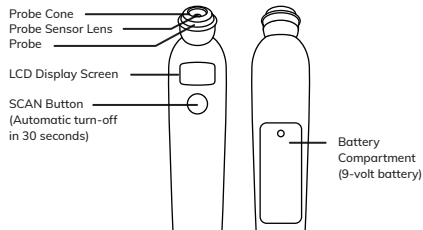TAKING A TEMPERATURE

Remove the protective cap.

Place probe flush on the middle of the forehead, press and hold the button.

Slide the thermometer in a straight line to the hairline while it beeps (don't touch glasses and/or hair).

Release the button and read the temperature.

SAFETY INSTRUCTIONS

- The TempoTherm™ thermometer is for home use only.
- Don't use over scars or open wounds.
- Don't drop or expose to extreme conditions.
- Don't try to sterilise the thermometer.
- Return it for service if needed (see address details on page 1).
- Avoid using near aerosols or fireplaces.

SCANNING

- Scan only the forehead and the exposed side of the head.
- Avoid measuring over hair, hats, or bandages.
- Wait 60 seconds between readings.
- Push aside blankets for accurate readings on infants.

ENVIRONMENTAL EFFECTS

- Let the thermometer adjust to room temperature for 30 minutes.
- Don't hold the thermometer by its head to avoid inaccurate readings.

DISPLAY MESSAGES

Scn means it's measuring.

HI indicates a very high temperature.

LO indicates a very low temperature.

HI A means: move the device to a cooler location.

LO A means: move the device to a warmer location.

Err may appear due to interference; wait or replace the battery. If Err messages continue, call or email Exergen, see page 1.

Blinking battery icon warns of low battery.

MAINTENANCE OF THE TEMPOTHERM™

- Keep the lens and cone clean with alcohol. Remove dust.
- Replace protective cap when not in use.
- Don't submerge it in water.
- Wipe with a clean cloth.
- Avoid abrasive cleaners.

FEATURES

- Meets accuracy standards for skin surface temperature.
- Works in ambient conditions 15,5°C to 40°C (60°F-104°F) with non-condensing humidity.
- Quick measurement in 1-3 seconds.
- Long battery life, depending on battery type.
- Compact and lightweight design.
- Impact-resistant casing.
- Storage range -20°C to 50°C (-4°F-122°F) with non-condensing humidity.
- Laboratory Accuracy: Display range 37°C to 39°C (98°F to 102°F) $\pm 0.2^\circ\text{C}$ ($\pm 0.4^\circ\text{F}$)*

*Meets ASTM E 1965-98 and EN60601-1 standards for electronic and radiation thermometers to the extent applicable to thermometers that measure the surface of the skin over the temporal artery.

IF YOU HAVE ANY ISSUE OR CONCERNS PLEASE CONTACT EXERGEN OR LOCAL COMPETENT AUTHORITY

+31 6 53226285
 TEMPOTHERM@EXERGEN.COM
 WWW.INTERNATIONAL.EXERGEN.COM
 EXERGEN CORPORATION
 400 PLEASANT STREET
 WATERTOWN, MA 02472, USA

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Emergo Consulting (UK) Limited
**MedEnvoy Global B.V.