


# EXERGEN TAT-5000S QR Cable Installation and Replacement



## Cable Installation

1. Remove cable from bag.
2. Align the cable and scanner as shown.  
(The scanner and cable are keyed.)
3. Insert the cable into the scanner until a click is heard.
4. Gently pull on the cable to insure proper insertion of the connector into the socket.


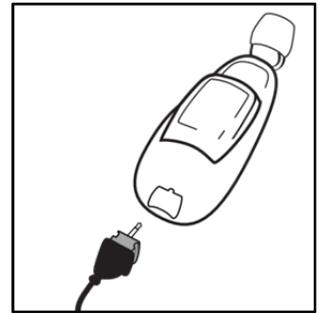


## Cable Replacement



1. Bend one leg of a paper clip and insert it into the hole in the side of the plastic housing. Push to release the battery cover, then move battery out of the way.

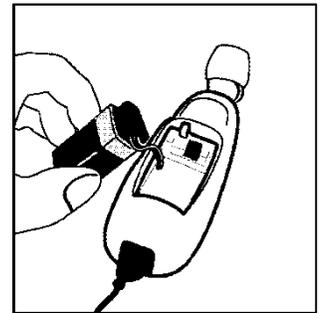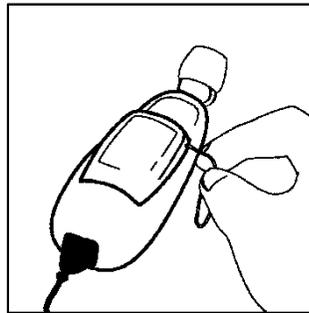

2. Depress black release button and remove cable while holding the release button down.

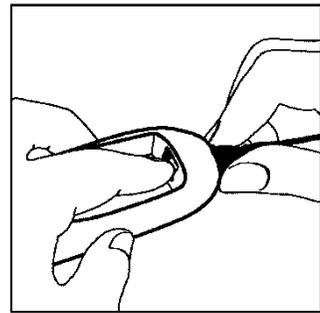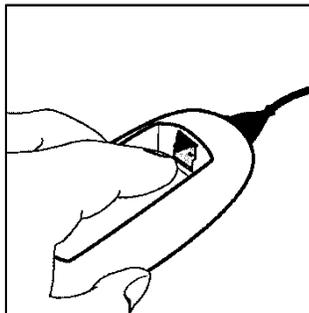

3. Orient cable plug properly for proper fit into the receptacle hole and install replacement cable.

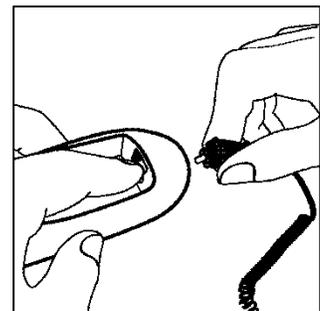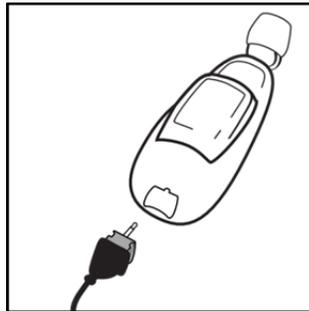

4. Replace the battery into the compartment. Re-install battery cover.

