


TAT-5000S Series Professional Models

EXERGEN
TemporalScanner™




Keyless Locking Wall Mount Installation and User Instructions

1. The mount is designed to operate with or without the locking feature. To enable the locking function, install the supplied locking mechanism using a Philips screwdriver as shown. For easier installation of the screw through the washer and latch, turn the Mount upside down. Seat the latch over the cylindrical standoff, then put washer on top, and install screw through the washer and latch into the plastic hole. Avoid overtightening. Excessive force may crack or strip the plastic.

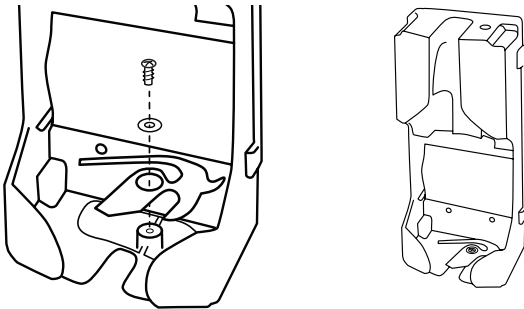


2. Use insert to properly secure all TAT-5000S models.

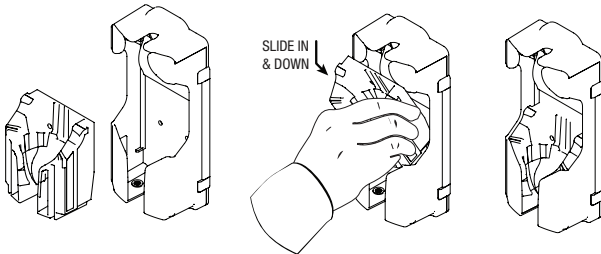


3. Align wall Mount device to wall and mark hole locations. Add insert to the wall mount locking device and securely attach to wall.


4. Install the Instrument into the Mount as shown by guiding the instrument through the opening by pushing the instrument into the locking groove with your thumb. The Locking Mechanism should make a click as it snaps.

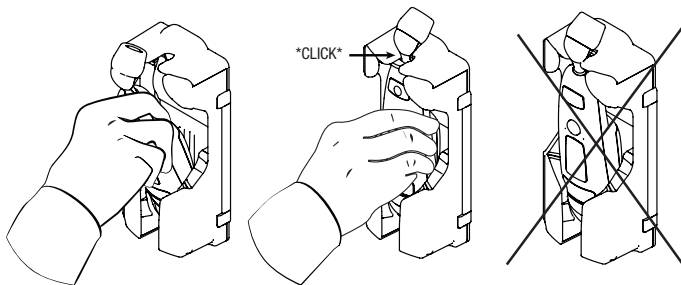


5. To remove the instrument, simply pull outwards on the top left flap as shown, Instrument will exit through the opening.

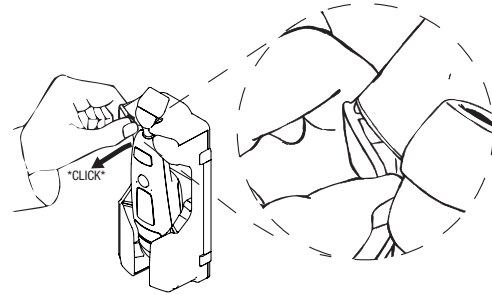


6. For Instruments with Steel Security Cable, securely attach threaded end into Mount, as shown.

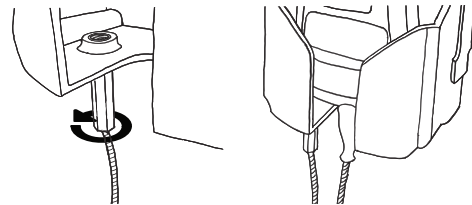

7. For Instruments with Coil Cord, securely attach threaded end into Mount, as shown.

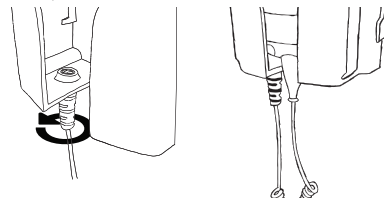

8. For Keyless Locking Wall Mounts with included cap holder, to replace caps, remove dispenser cover as shown and insert caps. Remove cap as needed by pulling downward on the bottom cap as shown.

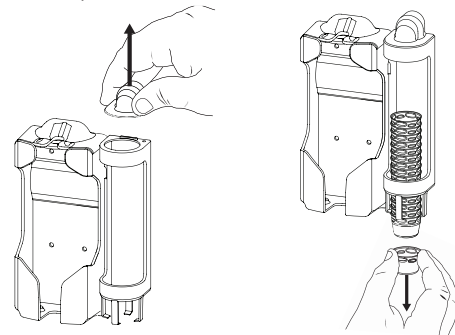

EXERGEN Corporation
400 Pleasant Street
Watertown, MA 02472



Invented, designed,
assembled, tested,
and packaged in the
U.S.A. by Exergen



Scan to Access Additional
Documentation and the
Latest Clinical Studies
www.exergen.com/s

©2026 Exergen Corp.
doc p/n 818672r4